

SONESTA S2 & S3

Maintenance & Cleaning

Maintenance

On a weekly basis, test the device for proper function and inspect cables for cuts and other damage. If in doubt, replace the relevant parts. The device should be cleaned routinely, as described below. Additional preventive maintenance is required according to the Service Manual.

Cleaning

- The unit needs to be unplugged from the wall power before any disinfection.
- Wipe all surfaces after each patient use.
- Please note that it is recommended to remove the seat cushion after each use and to wipe the surfaces below the seat cushion as well as the back of the seat cushion.
- Clean the chair with a dampened cloth and ordinary disinfectants. Follow established protocols for cleaning body fluids from the chair surfaces.
- Use any of the below listed recommended disinfectants on all surfaces. If they are not available, use any disinfectant with 1:10 dilution of bleach (5.25% sodium hypochlorite).

Recommended disinfectants:

- PDI Sani-Cloth SANI-CLOTH ® Bleach Wipe: Germicidal Disposable wipe.
- Clorox HealthCare Disinfecting Wipes ®



WARNING: Improper, or lack of cleaning after each use of the chair will cause damage to the appearance and performance.



WARNING: Iodophor type disinfectants (betadine, for example) may stain the cushions.



WARNING: Prolonged contact with bleach solutions with improper dilution ratios (for) in any cleaning agent may cause damage to both the appearance and performance.

